



Multiple Line Mobile Reader System for Lateral Flow Assays

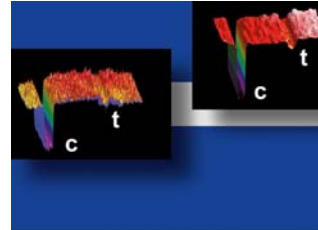

To turn on device press button on back side



- To start out testing make sure "TEST" is selected (It will show up with Black letters and white background). And press **Enter** ✓

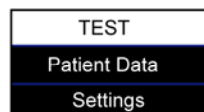

- Next Select Test by using the Up (2) or down Arrow (8) and then press **Enter**
- Next Select **Enter** to Assign "Analyte"
- Select Analyte and press **Enter**
- Next either type in Patient/Sample ID or use barcode scanner to enter ID
 - You can enter patient information by selecting **Enter**, Down (8) then press **Enter** Again or select previous ID by pressing **Enter**, Down (8) , Down (8), then **Enter**.
- After Analyte assignment has finished select continue
- Next you can either Scan Operator ID, Select previous operator (**Enter** , Down [8]) or **Enter** new Operator.
- To start measurement place cassette in slot. And read instructions on Screen
- After measurement has been taken you can Print by pressing # or Save Data by selecting

To View in more detail the measurement you can press *

Once Pressed * you can switch Screens by pressing Up or Down

- To go to Main menu at any time you can select **ESC**